Terraillon France & Headquarters

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Viale Sarca 45
Milano - Italy
Tel: + 39 02 66114199

Terraillon Asia Pacific Ltd

4/F, Eastern Centre
1065 King's Road
Quarry Bay - Hong Kong
Tel: + 852 2960 7288

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English

In accordance with the European directive 2002/96/CE, this product must not be disposed of with your other household waste. Dispose of your waste equipment by handing it over to a designated collection point for the recycling of waste electrical and electronic equipment. The separate collection and recycling of your waste equipment at the time of disposal will help to conserve natural resources and ensure that it is recycled in a manner that protects human health and the environment.

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SPEAKING ELECTRONIC WEIGHING SCALES

AUTOMATIC ELECTRONIC BATHROOM SCALE “LITHIUM”

Your new Terraillon electronic bathroom scale was designed to accurately display your weight. It should work for many years when used in normal conditions.

RECOMMENDATIONS FOR USE

- Always weigh yourself with the same bathroom scale on a hard surface. Avoid rugs and carpets.
- **Clean your bathroom scale with a damp cloth but do not allow the water to enter into the scale.**
- Remove the batteries if you do not intend to use your bathroom scale for an extended period.

POWER SUPPLY

These weighing scales use the following batteries, supplied with the appliance :

- 4 x 1.5V AAA batteries.
- Open the 1.5V battery compartment underneath the appliance. Place the batteries respecting the polarities (+/-) and close the lid.

Don't throw away worn out batteries. Place them in a container intended for this purpose so that they can be collected and recycled.

OPERATING INSTRUCTIONS

- *Adjustment of volume* : there are 3 volume levels: loud, quiet or silent. Press the button with a pen to select the desired volume.
- *Choice of language* : Press the button on the back of the weighing scales to select the language: French, English, German or Spanish.
- If you wish to change the measuring units from Metric to Imperial or vice versa (i.e. Kilograms to Stones to Pounds), simply adjust the conversion switch located on the under-side of the scales.

When the scales are first turned on or after replacing the batteries :

Give a brief tap in the centre of the appliance. The screen displays a row of zeros then switches off. Wait for 10 seconds before getting on the scales.

MEMORY FUNCTION

This model has 4 buttons each marked from 1 to 4. Each of the 4 buttons is intended for one different member of the household who can check his/her weight loss or gain between 2 measurements.

Memorising your weight

Whenever you weigh yourself using one of the buttons, your weight is automatically memorised.

SPEAKING ELECTRONIC WEIGHING SCALES AUTOMATIC ELECTRONIC BATHROOM SCALE “LITHIUM”

Weight gain or loss

The readout will display your weight gain or loss after you have been weighed.

1. Lightly press your personal key (e.g. memory 1) to start the appliance in memory mode. Memory number 1 appears. The scales display “Memory 1”.
2. When the screen indicates “0.0”, the scales tell you: “Ready / Stand on the platform”: stand on the platform and stand still until the figures are stabilised.
3. Your weight is displayed and the scales indicate the result: “you weigh ... kg”
4. Step off the platform.
5. After a few seconds, the difference between your current weight and last saved weight appears alternately on the screen.
If the minus sign is displayed in front of the figures, you have lost weight and your scales display “you have lost ... kg”.
If no sign appears in front of the figures displayed, you have gained weight and the scales display “you have gained ... kg”
6. Your weight is then displayed for a further 2 seconds and the scales are switched off automatically.

AUTOMATIC SWITCH OFF

When you get off your weighing scales, the screen displays a row of zeros, then the scales switch off automatically after a few seconds.

SPECIAL DISPLAYS

Display	In the case of:
<i>Err</i>	The scales display “error”: the weight difference is above or equal to 40 kg.
<i>Err0</i>	Incorrect initialisation: begin again as explained above.
<i>Err1</i>	Unstable weight: begin again.
<i>Err2</i>	Overload: remove weight immediately.
<i>Batt</i>	Battery failure: replace it.

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